


SEVEN


Surgical Instruments Kit Wall Chart

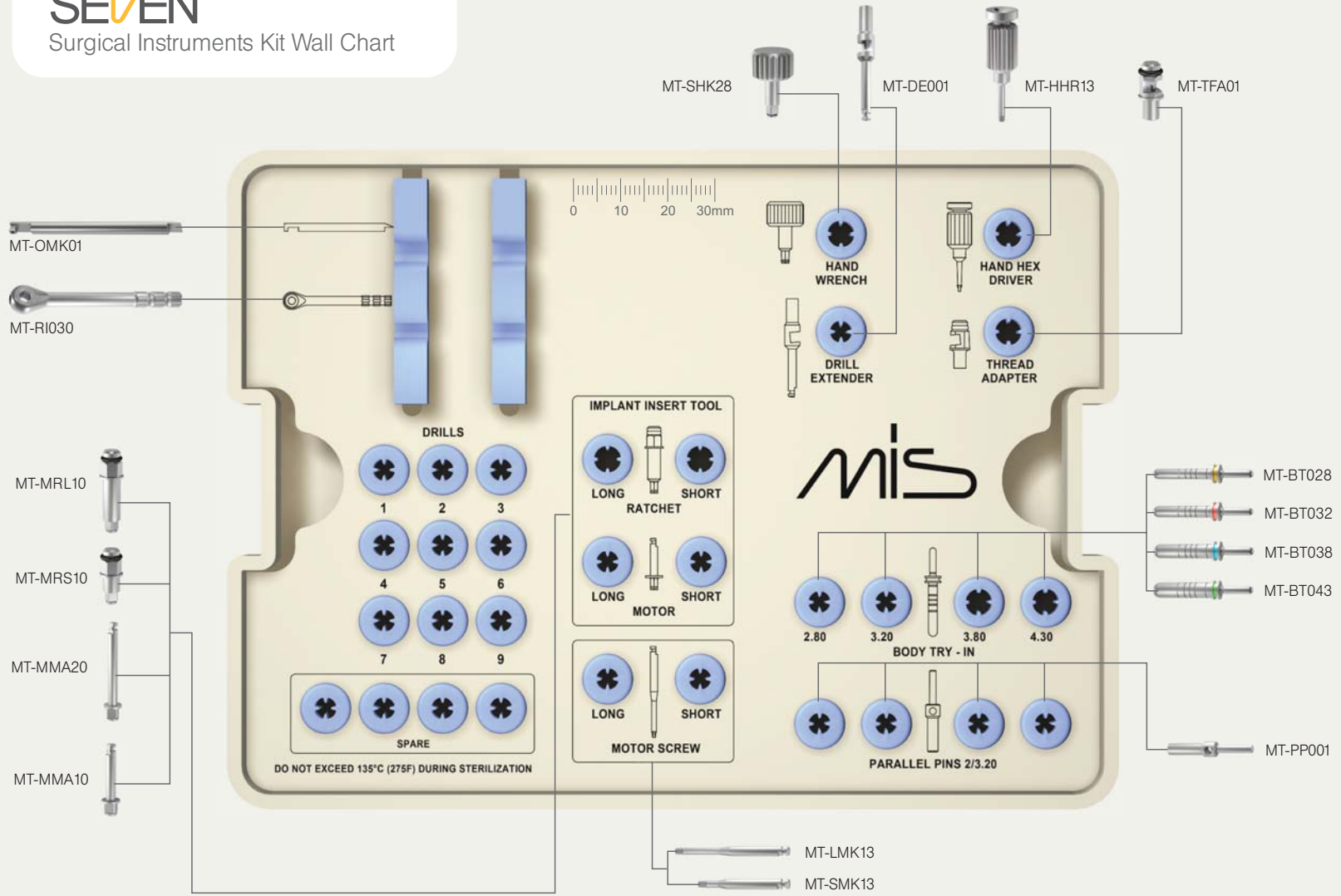

Implant Ø3.75mm Procedure

Recommended to use the final drill in bone type 1 and 2

Drill Speed (RPM)	1200-1500	900-1200	500-700	300-500	15-25
Diameter	Ø 1.90	Ø 2	Ø 2.80	Ø 2.80-Ø 3.60	Ø 3.75

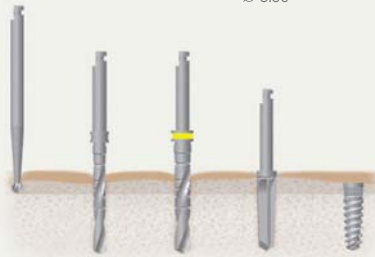

Procedure for bone type 3 and 4

Drill Speed (RPM)	1200-1500	900-1200	500-700	200-500	15-25
Diameter	Ø 1.90	Ø 2	Ø 2.80	Ø 3.75	Ø 3.75



Implant Ø4.20mm Procedure

Recommended to use the final drill in bone type 1 and 2

Drill Speed (RPM)	1200-1500	900-1200	500-700	400-700	200-400	15-25
Diameter	Ø 1.90	Ø 2	Ø 2.80	Ø 3.20	Ø 3.30-Ø 4.20	Ø 4.20



Procedure for bone type 3 and 4

Drill Speed (RPM)	1200-1500	900-1200	500-700	400-700	200-500	15-25
Diameter	Ø 1.90	Ø 2	Ø 2.80	Ø 3.20	Ø 4.20	Ø 4.20



Implant Ø5mm Procedure

Recommended to use the final drill in bone type 1 and 2

Drill Speed (RPM)	1200-1500	900-1200	500-700	400-700	400-600	200-400	15-25
Diameter	Ø 1.90	Ø 2	Ø 2.80	Ø 3.20	Ø 3.80	Ø 4.10-Ø 4.90	Ø 5



Procedure for bone type 3 and 4

Drill Speed (RPM)	1200-1500	900-1200	500-700	400-700	400-600	200-500	15-25
Diameter	Ø 1.90	Ø 2	Ø 2.80	Ø 3.20	Ø 3.80	Ø 5	Ø 5

