

## Unimatrix

Preformed sectional matrices system for Class II restorations.

Easy, quick and comfortable placing by a ring system, which replace the typical matrix holders;

### Directions for use

Choose the size of matrix which best fits with the cavity, place the matrix, install the wedge, and with the help of any clamp/forceps install the fixing ring; this ring will accomplish two functions, holding the matrix in the desired position and also by pushing the adjacent tooth; When the filling of the cavity is concluded, first remove the ring, then the wedge, and finally the matrix.

### Method of sterilization

The rings can be sterilized by any method.



- 4122A Unimatrix Kit - 25 assorted matrices + 1 Ring + Ring Plier **Rs. 1200/-**
- 4121 Unimatrix Refill - 100 matrices (L/M/S) **Rs. 980/-**

## Matrices

Transparent/Metallic Matrices with fixing system integrated

Transparent/Metallic Matrices with own fixing system.

Easy to place due its shape and thickness of 0,05mm;

Simple to tight to the tooth by squeezing its soft aluminum ring holder; Indicated for resins, glass ionomers and amalgams in Class II procedures.



- 4102 Premolar Transparent Matrices Pack of 20 **Rs. 780/-**
- 4103 Molar Transparent Matrices - Pack of 20 **Rs. 780/-**
- 4106 Deciduous Molar Transparent Matrices Pack of 20 **Rs. 780/-**
- 4107 Premolar Metallic Matrices Pack of 20 **Rs. 520/-**
- 4108 Molar Metallic Matrices -Pack of 20 **Rs. 520/-**

## Polyester Matrix Strips

Transparent Matrix Strip.

- Indicated for protecting adjacent teeth while the acid conditioning is being done;
- Due its transparency, resins can be cured through it;



- 4146 Polyester Matrix Strip - 0,05mm X 10mm X 15m **Rs. 195/-**
- 4136 Polyester Matrix Strip - 200 strips of 0,05mm X 10mm x 100mm **Rs. 300/-**

## Cervical Matrix

### Directions for use

- Adjust the matrix to the handle;
- Insert the last layer of composite;
- Compress to the matrix against the tooth forcing the composite to flow to incisal region;
- Holding the instrument remove the excesses of composite from the edges of the matrix and light cure it for 10 seconds through the matrix;
- Remove the matrix and complete the photopolymerization according to the recommendations of the manufacturer of the composite.



### Method of sterilization

The instruments and matrices are autoclavable.

- 4210 Cervical Matrix - 2 instruments + 10 assorted matrices **Rs. 1500/-**
- 4220 Cervical Matrix Refill - 20 pieces (XS/S/M/L/XL) **Rs. 420/-**
- 4225 Cervical Matrix Refill - 50 pieces (XS/S/M/L/XL) **Rs. 780/-**



## Tofflemire Matrices



Contoured stainless steel strips.

- Their high ductility and a thickness of 0,05mm allow these matrices to be contoured and adapted to the tooth very easily;
- Available for molars, premolars and the N° 1 universal V shape.

## Metallic Matrix Strips



Stainless steel strips in rolls.

- Dead soft stainless steel strips, with a thickness of 0,05mm;
- Available in rolls of 3mts and 5mts length, in widths of 5mm, 6mm and 7mm.

- 4129 Tofflemire Matrices for Premolar / Molar / No.1 - Pack of 12pcs **Rs. 75/-**
- 4142 Metallic Matrix Strip - Dead Soft - 0,05mm X 5mm X 5mtr **Rs. 230/-**
- 4139 Metallic Matrix Strip - Dead Soft - 0,05mm x 6mm x 5mtr **Rs. 230/-**
- 4143 Metallic Matrix Strip - Dead Soft - 0,05mm x 7mm X 5mtr **Rs. 230/-**

# Crowns

## Transparent crowns for teeth reconstruction.

- Indicated for provisory or permanent restorations;
- Applicable with composites and acrylic resins;
- Available for anterior and posterior teeth, for adults and deciduous.



### Pediatric transparent Crowns

They simplify the teeth reconstruction procedures to be done with photo-polymerizable resins.

Due their shapes and sizes; the requirement of adjustments for obtaining a perfect anatomy are just a few.

The pediatric crowns are offered in three sizes for each upper anterior tooth and in 2 sizes for the posterior ones.

### For Permanent Teeth

The permanent teeth crowns are offered in three sizes for each tooth for the anterior region and in 2 sizes for the posterior region.




4151	Crowns - Anterior teeth for adults - 64 pieces	Rs. 1150/-
4152	Crowns - Posterior teeth for adults - 64 pieces	Rs. 1150/-
4153	Crowns - Ant. & post. teeth for deciduous - 64 pieces	Rs. 1150/-
4154	Crowns - Anterior teeth for deciduous - 54 pieces	Rs. 1050/-
4156	Crowns Refill - Box containing 5 pieces (same size and n° of tooth)	Rs. 85/-

# Elastic Wedges

## Interdental Elastic Wedges.

- Compatible with all matrix systems for Class III restorations;
- Act simultaneously at vestibular and lingual regions, providing a perfect adjustment of the matrix to the tooth;
- Promote a tooth separation without harming the gum, allowing to get a tight interproximal contact point;
- Available in three sizes codified by colors.

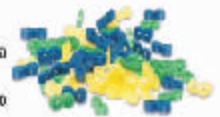

### Directions for use



- In order to attain a better handling of the Elastic Wedge, it is recommended to use porta clamps type tweezers;
- Stretch the Elastic Wedge using the tweezers;
- Place the matrix and then the Elastic Wedge;
- Place the ring behind the Elastic Wedge if necessary;
- After the ring was removed, the Elastic Wedge should be cut with scissors while is being stretched from one of its extremities.



4801	Elastic Wedges - 75 assorted pieces (three sizes)	Rs. 550/-
4803	Elastic Wedges - 25 pieces X 2,0mm	Rs. 205/-
4804	Elastic Wedges - 25 pieces X 2,6mm	Rs. 205/-
4805	Elastic Wedges - 25 pieces X 3,2mm	Rs. 205/-

# Contact + Gold

## Instruments for contact point reconstruction.

- Translucent Instruments that allows to light cure through their tips while the matrix is being hold attached to the adjacent tooth;
- One Instrument for large cavities (L) and one for small cavities (S);
- Each Instrument has a distal and a mesial tip.



### The instruments are autoclavable

#### Directions for use

Before using the Contact - Gold, the cavity should have at least one increment of resin properly adjusted and polymerized;

Introduce a small quantity of composite into the cavity, enough to cover the tip of the Instrument;

Introduce the Instrument into the resin in the cavity and apply a moderate force until the matrix gets in contact with the adjacent tooth; keeping it in the contact point position, photo-polymerize the resin through the instrument. When the resin is cured, release the pressure and remove the instrument from the cavity;

Complete the filling of the cavity with the conventional procedure.



4202G	Contact + Gold - 2 instruments for contact points reconstruction	Rs. 1500/-
-------	--	------------

## Anatomic Wedges

### Anatomic wooden wedges.

- Anatomically shaped for a better adjustment to the interproximal spaces and to separate adjacent teeth;
- Available in 5 sizes codified by colors.



## Interdental Wedges

### Interdental wooden Wedges.

- Symmetric geometry and sharp angles;
- Lower level at the posterior margin, which facilitates their handling with tweezers;
- Available in 4 sizes, codified by colors.



## Reflexive Wedges

### Transparent wedges.

- Cure through wedges that in combination with transparent matrices allows light curing composites on interdental cervical areas;
- Available in 2 sizes.



4105	Translucent plastic Wedges - assortment of 100 pieces	Rs. 350/-
4953	Anatomic wooden wedges for deciduous - 100 pieces n° 1	Rs. 225/-
4957C	Anatomic wooden wedges - assortment of 100 pieces - colored by sizes	Rs. 225/-
4959C	Interdental wooden wedges - assortment of 100 pieces - colored by sizes	Rs. 225/-

# Fotoplus

## Light cure through instruments.

- Developed to prevent a marginal leakage on resins Class II restorations; providing a good marginal sealing;
- Translucent instruments that allows to light cure through their tips while the composites are being held under pressure on the bottom of the cavity;
- 2 instruments, with 4 different tips.



### Directions for use

- Choose the tip that fits better with the shape of the cavity in treatment; place the first increment of resin into the cavity, and keep it under a low pressure while it is photo-polymerized;
- When the composite is cured, release the pressure and remove the instrument from the cavity.

### Method of sterilization

The Fotoplus instruments are autoclavable.

4201 Fotoplus - 2 instruments to condense composites **Rs. 750/-**

# Esthetic Plus

## Silicone brushes for composites shaping.

- Anti-adherent silicone brushes for composites sculpting;
- Avoid the lines and marks existing between the composites layers caused by the stratification technique;
- Get smooth surfaces with a significant time reduction for the posterior finishing and polishing;
- Hard and soft tips in order to find the best compatibility with the hardness of the composite in use



### Directions for use

- Select the instrument and brushes (tips) which are more suitable to the requirements of the cavity to be restored;
- Fit the brushes onto the ends of the instrument;
- Sculpt the composite layer by layer by applying soft touches.

### Method of sterilization

Instruments and brushes are autoclavable

4206 Esthetic Plus - 2 Instruments + 8 brushes (tips) **Rs. 1600/-**

4204 Esthetic Plus refill - 4 soft brushes (tips) **Rs. 470/-**

4207 Esthetic Plus refill - 4 hard brushes (tips) **Rs. 470/-**

# Microcut

## Frame to which can be adapted saw blades and diamond strips.



- Saw blade for removing excesses of any type of filling materials from the interproximal areas without breaking the contact point due its thickness of only 0,05mm;
- Diamond strips for stripping in orthodontic procedures;
- Safe for the patient and very easy to handle;
- All components are autoclavable.



# Manual Microcut

## Saw blade for removal of excesses in the interproximal regions.

- Manual handling saw blade strip for removing excesses of any type of filling materials from the interproximal areas without breasing the contact point relation due its thickness of only 0,05mm.
- Autoclavable.



3030 Microcut - 1 Hack-saw frame + 1 saw blade + 1 diamond strip **Rs. 1200/-**

3031 Microcut Refill - 5 saw blades **Rs. 375/-**

3032 Microcut Refill - 5 diamond strips **Rs. 520/-**

3033 Microcut Manual - 1 serrated steel blade 4mm X 150mm **Rs. 200/-**

# Optimize

## Composite finishing and polishing silicone tips.

- One single tip is enough to accomplish the finishing and polishing of the composite restoration, saving time and abrasive materials during this operation;
- High resistance tips that can be used several times;
- Available in 3 shapes.



### Directions for use

- Fix the mandrel to the contra-angle and attach the selected tip to the free end of the mandrel.
- Optimize tips should be used on dry conditions with a low to medium rotation speed, applying touches at intermittent intervals.

### Method of sterilization

The mandrel and tips are autoclavable.



3021 Optimize - 24 assorted pieces + mandrel **Rs. 900/-**

3022S Optimize cone/Disc/Cup Refill 5 Pes **Rs. 180/-**

# Fiberglass

## Multipurpose Fiberglass burs.



- Indicated to remove cement, composites, adhesives and stains, without risk of harming the enamel, ceramics, metals and soft tissues;
- They can be used in orthodontics, periodontology, implantology, and clinical procedures in general;
- Continuous abrasive effect, and self-sharpening capacity;
- They can be autoclaved and reused multiple times;

### Available in 6 different shapes.



#### Applications:

- Use with Low speed with a water spray
- Elimination of adhesive cement after brackets removal
- Removal of stains without harming the dental enamel
- Removal and finishing of composites without damaging the enamel;
- Cement removal after sealing of ceramic bridges
- Root planning after in depth cleaning for a better flap reattachment;
- Elimination of the remnant sealing cement from the implant carrying prosthetic components.



3100 Fiberglass - 6 assorted burs (Individual Size) **Rs. 1110/-**

3101 Fiberglass Bur individual size 1 to 6 Each **Rs. 325/-**

# Praxis

## Finishing and polishing discs for composites.

Pop on mandrel system, easy and fast for the installation of the discs as well as the inversion of the abrasive surface to mesial and distal positions;

Ultra fine discs very flexible and versatile;

Four grits codified by colors, in sizes of 1/2" and 3/8".



#### Directions for use

- Fix the mandrel into the contra-angle and attach the disc on the free end of the mandrel by pressing it;
- For a better result of the polishing and finishing procedure, it's recommended to use the four grits progressively, starting with the coarse grit and ending with the extra-fine one;
- Praxis discs should be used dried, in a low speed rotation by applying light touches at intermittent intervals.



3002 Praxis - 60 assorted discs + pop on mandrel **Rs. 1100/-**


3009 Praxis Refill 3/8" - 10 discs of any grit (G/M/F/XF) **Rs. 95/-**

3010 Praxis Refill 1/2" - 10 discs of any grit (G/M/F/XF) **Rs. 95/-**

# Superfix

## Finishing and polishing discs for composites.

- Versaplex mandrel system that works by pressure, it doesn't have metallic core, what permits to use all the abrasive surface;
- Ultra fine discs, very flexible and versatile;
- Four grits codified by colors, in sizes of 9,5mm and 12,7mm.

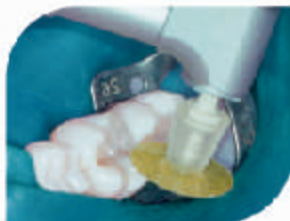

#### Directions for use

- Fix the mandrel into the contra-angle and connect the free end of the mandrel into the flexible support of the disc.
- For a better result of the polishing and finishing procedure, it's recommended to use the four grits progressively, starting with the coarse grit and ending with the extra-fine one;
- Superfix discs should be used dried, on a low speed rotation with light touches at intermittent intervals.



3015 Superfix - 50 assorted pieces (2 size & 4 Grits) + mandrel **Rs. 950/-**

3012 Superfix Refill 25 pieces (Grits G/M/F/XF) **Rs. 540/-**

# TDV Sandpaper Discs

## Finishing and polishing discs for composites.

- Ultra-fine discs, very flexible and versatile;
- Excellent cost benefit relation;
- Four grits codified by colors, in sizes of 12mm, 16mm and 19mm.



#### Directions for use

Install the desired disc on a **conventional mandrel**;

In order to achieve a better result, use the discs in the 4 granulations, successively from the coarsest to the finest one;

The product must be applied dry on low to medium speed by intermittent touches.



3026 Sandpaper Discs 16mm - 50 discs (G / M / F / XF) **Rs. 195/-**

3027 Sandpaper Discs 12mm - 50 discs (G / M / F / XF) **Rs. 195/-**

3028 Sandpaper Discs 19mm - 50 discs (G / M / F / XF) **Rs. 195/-**

# Abrasive Steel Strips

## Abrasive Steel Strips



Stainless steel strips; aluminum oxide coated.

- For amalgam finishing;
- Only 0,10mm thickness;
- Available in widths of 4mm and 5mm;
- Autoclavable.



## Diamond Strip



Stainless steel strips; diamond coated.

- Indicated for orthodontic stripping, slices, and the finishing of filling materials;
- Available in 2,5mm width;
- Autoclavable.



3051 Abrasive Steel Strips - 12 strips of 4mm X 150 mm **Rs. 265/-**

3052 Abrasive Steel Strips - 12 strips of 5mm X 150 mm **Rs. 360/-**

3053 Diamond Strip - 12 strips - 2,5mm X 110mm **Rs. 300/-**

# Approximal Finishing Sandpaper Strips

Polyester strips; aluminum oxide coated.

- For finishing and polishing composites;
- Medium and fine grits separated by a neutral space for an easier insertion through out the contact point area without harming it;
- Available in 2,5mm and 4,0 mm widths.



### Directions for use

- Introduce the strip into the interproximal space through the neutral space situated on the middle of the strip;
- Carry out the approximal face polishing with back and forth movements, starting with the medium granulation and finishing with the fine granulation side.



3065 Approximal Finishing Strips - 100 pieces of 2,5mm X 170mm **Rs. 240/-**

3066 Approximal Finishing Strips - 100 pieces of 4mm X 170mm **Rs. 240/-**

# Polimax

Impregnated felt wheels.

- Superb final polishing of composites and shiny surfaces;
- Impregnated with ultra fine abrasive dehydrated polishing paste;
- Activated with a few drops of natural water;
- Get beautiful white and "bright" teeth after whitening procedures.



### Directions for use

- Adjust the Versaplus mandrel into the hand piece;
- Fix the Polimax disc on the mandrel by introducing the free end of the mandrel into the flexible support situated on the back side of the disc;
- Moisturize the felt disc with a few drops of water and apply it on a low rotation with light and intermittent touches.

3015 Polimax - 24 impregnated felt wheels 9,5mm + mandrel **Rs. 950/-**

3048 Polimax - Refill Pack of 5 Pos **Rs. 180/-**

# Polifix

Velvety discs.

- Perfect brightness on all type of surfaces;
- Very soft and smooth velvety surface;
- Applicable with all type of polishing pastes;
- Versaplus mandrel system, easy, fast and safe.



# Felt Discs

FELT DISCS  
Natural wool wheels.

- Perfect brightness on all type of surfaces;
- Applicable with all sort of polishing pastes;
- Get beautiful white and "bright" teeth after whitening procedures;
- Available in sizes of 8mm and 12mm.



IMPREGNATED FELT DISCS  
Natural wool impregnated wheels.

- Superb final polishing of composites and shiny surfaces;
- Impregnated with ultra fine abrasive dehydrated polishing paste; activated with a few drops of natural water;
- Get beautiful white and "bright" teeth after whitening procedures;
- Available in 12mm size.



3049 Polifix - 24 velvety discs + mandrel **Rs. 610/-**

3041 Impregnated Felt Wheels - 24 wheels 12mm **Rs. 770/-**

3043 Felt Wheels 12mm - 12 wheels **Rs. 245/-**

3044 Felt Wheels 8mm - 12 wheels **Rs. 245/-**

# Poligloss

## Aluminum oxide polishing paste.

- To obtain a high gloss and an excellent natural brightness on composites;
- Extra-fine grit (1 to 2 microns);
- Hydro-soluble;
- Easy cleaning.



### Directions for Use

- Poligloss polishing paste application should be done with a felt disc or a velvety disc for obtaining the best result on the final polishing of the restorative material;
- Install the mandrel and the felt disc on the contra-angle, add one drop of Poligloss on the disc, and obtain the best gloss by doing soft and intermittent touches onto the tooth.



3042 Poligloss - syringe with 2g of polishing paste + Mandril + Felt Disc 8mm - 2 Nos + 12mm - 2Nos. **Rs. 420/-**

# Diamond Gloss

## Diamond polishing paste.

- To obtain a high gloss and an excellent natural brightness on all restorative materials and enamel;
- Useful at the clinic and the laboratory;
- Extra-fine grit (1 to 2 microns);
- Hydro-soluble, easy cleaning.



### Directions for use

- Diamond Gloss polishing paste should be used together with felt discs or velvety discs as POLIFIX system; which potentiates its abrasive action in the final polishing of restorative materials;
- Add the required quantity of Diamond Gloss on the disc and apply with intermittent touches.



3040 Diamond Gloss - syringe with 2g of diamond polishing paste + Mandril + Felt Disc 8mm - 2 Nos + 12mm - 2Nos. **Rs. 550/-**

# Prophylactic Strips

## Stainless steel strips for tartar removing.

- Easy, fast and safe to remove tartar without damaging the dental enamel;
- Not abrasive, tartar is removed by a mechanical action provided by the micro-crests located on the surface of the strip;
- Easy access to the interdental area due its thickness of 0.05mm.



### Directions for use

- Insert the strip into the teeth gap, taking care that the rough side is turned to the desired tooth;
- Hold the strip by its extremities, pulling it against the tooth and applying it in to and fro movements until the desired effect is attained.



3076 Prophylactic Strip - 10 perforated steel strips for tartar-removing **Rs. 445/-**

# Profilax

## Prophy cups and points.

- Due its 100% silicone composition it's harmless for the gingiva;
- Versaplex mandrel system, designed with a hand piece protection.



### Directions for use

- Introduce the end of the mandrel into the contra-angle and fit the required tip onto the other end of the Versaplex mandrel; and then, after adding the prophylactic paste, just use the tip on a low speed at intermittent intervals.



Method of sterilization  
Tips and mandrel are autoclavable.

3075 Profilax - 4 cups + 2 points + 1 mandrel **Rs. 170/-**  
5 cups/ 1 point **Rs. 75/-**

# Kits

## Photo Kit

### Selection of materials for aesthetic restorations.

Content:  
Unimatrix sectional matrices + fixing ring - Matrices with own fixing system in steel and polyester - Superfix polishing discs - Polimax felt discs - Versaplex mandrel - Assorted wooden Wedges - Cure through Wedges - Finishing sandpaper strips - Diamond strips - Transparent Crowns.

### Matrices Kit

Content:  
Unimatrix sectional matrices assorted - 2 fixing rings - 10 for molar and 10 for premolar in polyester + 10 for molar and 10 for premolar in steel - Assorted Wedges.



### Odontopediatric Kit

#### Selection of materials for aesthetic restorations on deciduous teeth.

Content:  
Unimatrix sectional matrices + fixing ring - Matrices with own fixing system in polyester - Superfix polishing discs - Polimax felt discs - Versaplex mandrel - Wooden Wedges - Cure through Wedges - Finishing sandpaper strips - Diamond strips - Transparent Crowns.

4109 Photo Kit - selection of materials **Rs. 1800/-**  
4110 Matrices Kit - assortment of matrices and wedges **Rs. 2340/-**  
6080 Odontopediatric Kit - selection of materials **Rs. 1860/-**

For Online Trading please log on  
[www.confidentonline.net](http://www.confidentonline.net)

## Confident Sales India Pvt.Ltd.

No. 47, Pete Channappa Indl.Este,  
Magadi Road, Kamakshipalya Bangalore - 560 079  
Ph : +91 80 23281368/376 Fax : +91 80 23482156  
info@confidentonline.com